Plataforma Dedicada à Síndrome do Olho Seco

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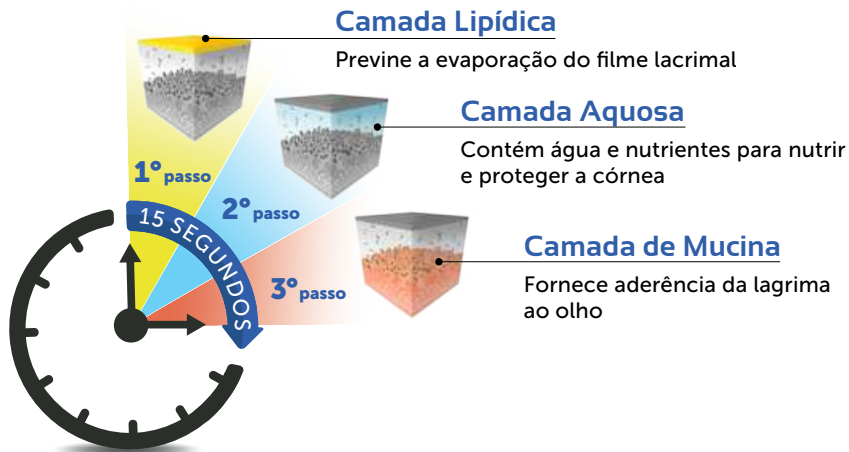
Ocular Surface Analysis – Dry Eye Suite

The fastest and most complete Dry Eye suite

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
The tests required for the diagnosis of the Dry Eye are well specified in the report of the DEWS II 2017 TFOS

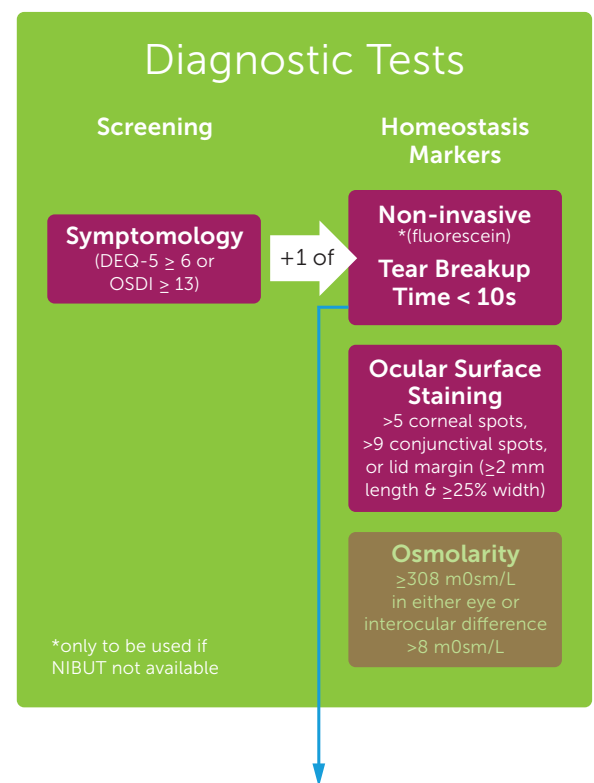Idra Full allows the complete analysis using the guidelines in a completely automatic and objective way, providing the results automatically without operator intervention, thus making the analysis incredibly fast and easily delegated.

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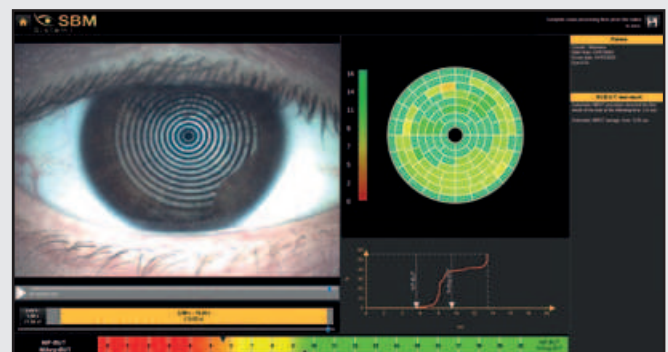
plus • full


Ocular Surface Analysis – Dry Eye Suite

Automatic Interferometry

Thanks to the anterior illumination module, Idrá can acquire the lipid layer secretion on the cornea.

The device highlights the lipid layer and the software evaluates the quantity and quality of the lipid component on the tear film.

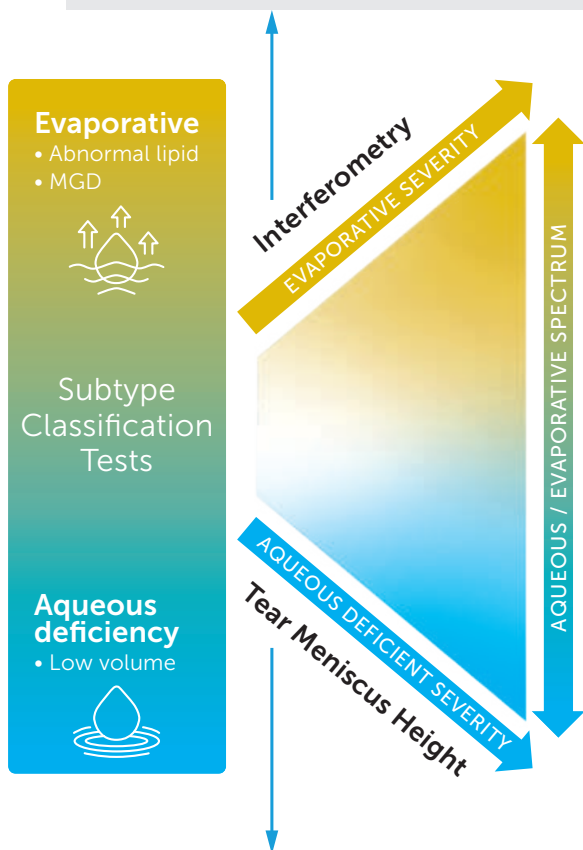basic • **plus** subjective • **full** automatic

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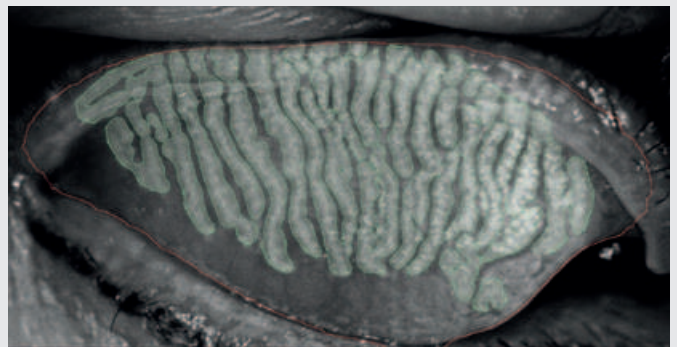
Automatic tear meniscus height

Low tear production may result in aqueous tear deficiency (ATD) and cause dry eye symptoms. However, measuring the tear volume is difficult since the methods normally used are invasive and irritating. Reflex tear production can be induced, giving an over-estimation of basal tear flow and volume.

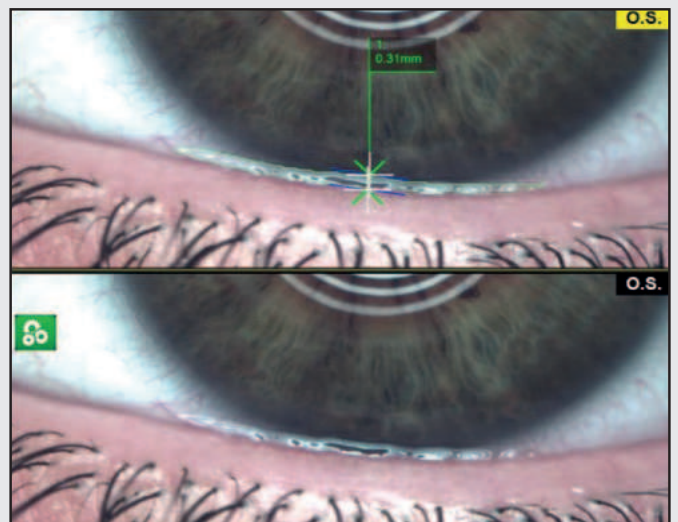
Tear meniscus height is related to the tear secretion rate and tear stability and for this reason it's a good indicator of tear production. The aqueous layer is evaluated through the non-invasive "Tear Meniscus" test and is then classified into different categories. The tear meniscus can be examined considering its height, regularity and shape.

basic subjective

An artificial intelligence determinates automatically:

- Position of tear meniscus
- Highest value in TM

plus • **full** automatic


Additional Evaporative Dry Eye exams

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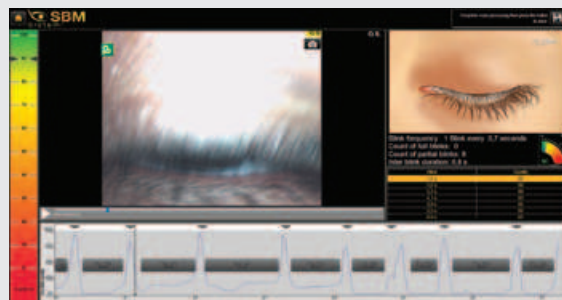
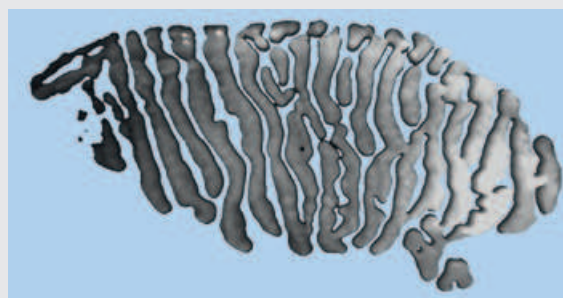
plus • full

Cylindrical Dandruff and Blepharitis

Blepharitis is an inflammation caused by some bacteria that lie at the base of eyelashes. They produce dandruff-like flakes in the skin, which lead to infection and inflammation.

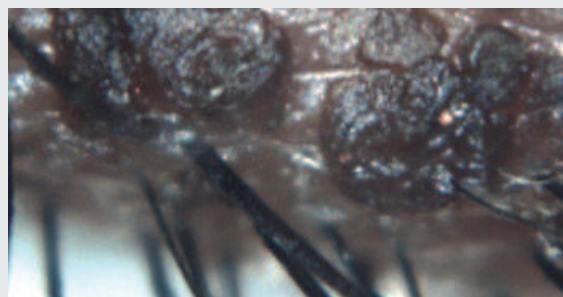Problems with the Meibomian Glands (meibomianitis) in the eyelids can also cause blepharitis.

Pupillometry

The measurement of the pupil diameter has become increasingly important in the field of refractive surgery.

Larger scotopic pupil sizes may be partially responsible for the occurrence of postoperative symptoms such as halos, glare, and monocular diplopia. Refractive surgeons also need an accurate scotopic pupil measurement to determine appropriate treatment zones for excimer laser, corneal, and intraocular surgery.

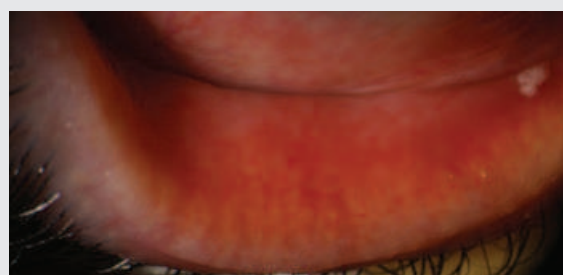
Demodex

Demodex brevis is a kind of mite found on the skin of humans.

D. brevis is so small that mites can't be seen macroscopically. The average mite causes noticeable reactions and problems in people largely infested. Symptoms of D. brevis usually only occur in case of large infestations.


Signs might include: Red skin, Rough or tough skin, Scaly or patchy skin. The symptoms of D. brevis are similar to those of D. folliculorum. The key difference is their location.

Additional Evaporative Dry Eye exams

Efron / CCLRU / Jenvis

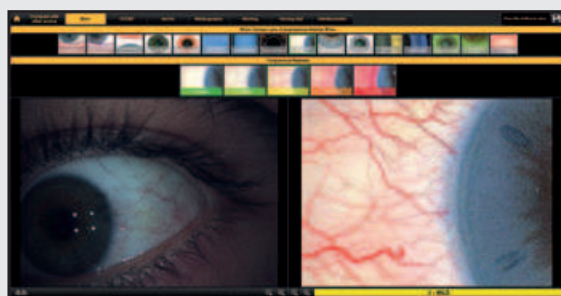
Comparative tables

Bulbar redness

Acquiring an image of the conjunctiva, it will be possible to compare the patient's condition with different international grading scales.

Once the image of the conjunctiva with its blood vessels is captured, it is possible to compare it with the classification sheets of bulbar and limbal redness degrees.

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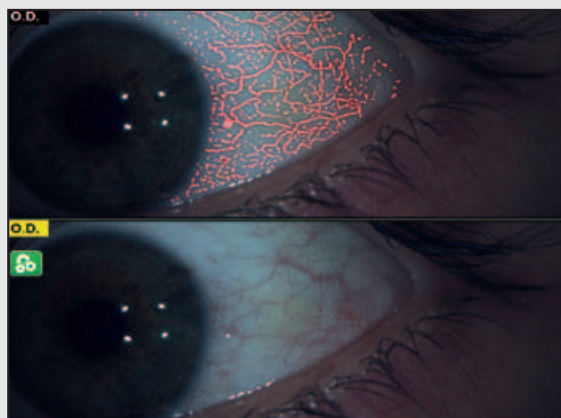Treatment protocol

SBM Sistemi software provides different solution to help MD/OD/users to assign therapy:

MANUALLY

Treatment managing

Through TREATMENT MANAGING tab, the software allows the physician to fill in the database with all drugs, integrators and treatments available in his practice.

Any treatment as vitamins, Omega-3, eye drops, hot packs and IPL/Radiofrequency, can be loaded on the software to prescribe the products of the brands that the doctor prefers.

AUTOMATICALLY

Automatic treatment suggestion

The unique integrated algorithm, developed in collaboration with MD. Luca Vigo from Studio Medico Carones, can provide a dedicated treatment approach based on the results of the analysis.

In addition...

All users can customize their own protocol adding treatment procedure to be chosen automatically right after performing the exams (this makes it possible as well to delegate the diagnosis to an assistant).

Each of the options provides comprehensive and dedicated printable reports.

Dry eye follow up

This is an app for the smartphone of the patient to receive exams results, therapy, and a complete knowledge about the dry eye disease.

plus · full

Benefits of Dry eye follow up

- Fast and easy doctor-patient interactive communication Platform creating a two-way communication;
- Accurate reports sharing and updates;
- Tracking of results and progress over time to show fluctuations in symptoms;
- Help in regularly scheduling appointments, interact with clinic/hospital;
- Sending automatic reminders to patients and notifications to doctors;
- Speeding up the process of providing information, booking appointments, and setting up the treatment procedures prescribed by the doctor.

Stop forgetting your drugs

Set automatically on your mobile phone all treatments.

The App will remind you when and how to use the specific suggested drug.

Download on

Description

Joystick One-click acquisition

The optional SBM Smart Base is built precisely, smooth, and high-quality. Images and movies can be captured instantly and conveniently by pressing the joystick button.

Left/Right automatic detection

The SBM Smart Base automatically recognizes the right and left eye, allowing an even faster diagnosis of the ocular surface.

www.sbmsistemi.com

Strada Torino, 43 - 10043 Orbassano (Torino) Italy - Tel. +39.011.19923378 - info@sbmsistemi.com